BI-GO

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Mobiel BV
Botter 9
2411 NM Bodegraven
The Netherlands

Contents

Before using the Bi-Go	4
The Bi-Go self-balancing mobility aid	5
• User purpose	
• Warranty	
Safety	7
• Guidelines	
• Safety guidelines	
• Remaining risks	
Technical specifications	8
Operation	9
• InfoKey	
• Use	
Maintenance	13
• Charging	
• Battery maintenance	
Maintenance tips	
Service	14
• Settings of the Bi-Go	
• Service	
• Tire pressure	
Liability	15

EN

Requirements to be met by the rider of the Bi-Go

The Bi-Go provides optimum safety and performance when the rider meets the following qualifications:

- Minimum age of 16 years old
- Minimum weight of 45 kg
- Maximum weight of 110 kg
- Maximum height of 190 cm
- Has sufficient strength in hands and fingers
- Good upper body control (trunk stability)

NL

Before using the Bi-Go

We ask your attention to the contents of this user manual. It contains information and instructions that are important for the proper and safe use of the Bi-Go and long term driving enjoyment. Please read all instructions carefully and contact us if you have any questions.

DE

Insurance

If you want to drive your Bi-Go on public roads, your Bi-Go must be at least Liability-insured. As owner of a Bi-Go, you are therefore legally obliged (just like a car) to insure your Bi-Go against third-party liability. Of course, you can expand the third-party insurance with:

- All-risk (in case of loss, theft and damage)
- Accident insurance
- Legal assistance

The insurer will ask you for your personal information and the chassis number. The chassis number is the 12-digit serial number of the Segway PT and 7-digit serial number of the Bi-Go. You will receive these serial numbers from your supplier upon delivery of the Bi-Go.

Batteries

To keep the batteries of the Bi-Go in optimal condition and prevent breakdowns, it is important that the Bi-Go is always on the charger at times when you are not using it. In addition, we recommend that you use the Bi-Go regularly (weekly) to keep the batteries in optimal condition.

The Bi-Go self-balancing mobility aid

The Bi-Go is intended as an assistive device for people with walking disabilities. The owner is responsible for ensuring that the user has read and understood this manual. The manual must be available for reference by the user at all times.

User purpose

The Bi-Go is intended as an aid for people with disabilities to get around indoors and outdoors. The Bi-Go can be used on paved and unpaved roads. It is prohibited to use the Bi-Go as a tractor. It is also prohibited to take (domestic) animals, children or a baby on your lap while operating the Bi-Go. If this does happen, the supplier and/or Mobiel BV will not be held liable of any form of injury and any form of warranty will be void.

Warranty provision

All damage to parts of the Bi-Go, within the warranty period, caused by material, manufacturing and/or construction errors, will be remedied as quickly as possible by your supplier. Your supplier grants this warranty to the original purchaser (as well as the purchaser of the product, as described below) of an original Bi-Go replacement parts and accessories obtained from your supplier.

The term original purchaser, as used in these warranty terms, means the first purchaser who purchases an original Bi-Go, replacement parts or accessories from your supplier. The term purchaser, as used in these warranty terms, means the person who purchased the Bi-Go, replacement parts or accessories within the warranty period and notified Bi-Go in writing of the purchase of a Bi-Go, parts or accessories within 10 days of purchase.

Subject to exclusions and conditions of warranty described below, your supplier will warranty the Bi-Go Powerbase against defects in materials and workmanship for a period of 1 year after delivery to the original purchaser.

Subject to the exclusions and conditions of warranty described below, your supplier will warranty all other components of the Bi-Go, including: grips, trims (including wheel covers, emblem, front and rear covers, gearbox covers), mats, wheels, tires (including tube), infokey Controller, infokey dock, fenders, charging cord, replacement parts and accessories against defects in materials and workmanship for a period of 90 days after delivery to the original purchaser.

Replacement parts refers to all replacement parts purchased separately for a Bi-Go regardless of whether a part original is covered by 1 year warranty or a part original is covered by 90 days warranty.

EN

Accessory refers to a product sold by the supplier for use with a Bi-Go. The only accessories covered by this warranty are accessories manufactured by Mobiel BV. If an accessory sold is not manufactured by Mobiel BV, the manufacturer's warranty provisions will apply.

NL

During the warranty period, your supplier will within a reasonable time, repair or replace (with a new or refurbished product with the same specifications and up-to-date software, if applicable) the defective component of the Bi-Go, replacement parts or accessories. Your supplier may simultaneously replace non-faulty components if they are part of a larger part/component of which the faulty component is a part. All replaced components, parts and accessories are the property of your supplier.

DE

The warranty is immediately voided if:

These warranty provisions do not cover damage to a Bi-Go, replacement parts and accessories caused by the following:

- Any external causes such as (without limitation) falls, accidents, collisions, fire, immersion in water, freezing or collision with objects;
- Misuse of the product such as driving over obstacles, overloading, racing, or use of the Bi-Go other than described under the heading user purpose, replacement parts and accessories that is contrary to the user manual or instructions for use;
- Changing or modifying the Bi-Go, replacement parts and accessories;
- Opening or loosening the console cover plate of the Bi-Go or accessing parts inside the Powerbase (controller board, balance sensors, motors and internal wiring);
- Damage to the stand caused by stepping on, or moving the Bi-Go with the stand deployed (in use);
- Damage to the paint, trims and appearance of the Bi-Go caused by weather or other external elements;
- Cleaning with a pressure washer or cleaning agents;
- Exposing it to extreme environmental conditions;
- Failure or poor maintenance of the Bi-Go, replacement parts and accessories;
- Damage caused by non-Bi-Go products or components;
- Failure to charge the Bi-Go correctly;
- Driving the Bi-Go by a person under 16 years of age, lighter than 45 kilograms or heavier than 110 kilograms.

Safety

Please read this manual before using the Bi-Go, or before performing any maintenance or repair work. When in doubt about work, you can always consult your supplier. Mobiel BV or the supplier is not liable for damage to people working with, on or near the Bi-Go, caused by the use of non-original parts and accessories, carrying out repairs and settings incorrectly, operating or cleaning the Bi-Go incorrectly.

Guidelines

Mobiel BV has done everything possible to make the Bi-Go as safe as possible. The design of the Bi-Go has taken into account requirements of the European Machinery Directives 2006/42/EC and 93/42/EEC. When designing and setting up the machines, the design department of Mobiel BV has taken into account and tried to anticipate as much as possible expected risks when using the machines.

Safety regulations

- Do not use the Bi-Go without training/instructions from your supplier;
- Never use the Bi-Go before you have read and understood the manual;
- When in doubt about use, always contact your supplier;
- It is recommended that you wear a helmet while using the Bi-Go;
- Always turn on the lights when it is dark or at dusk;
- Never use the Bi-Go on slopes steeper than 15%;
- It is strictly forbidden to remain seated in the Bi-Go during transportation by, for example, a wheelchair cab;
- It is forbidden to take children/baby on your lap while driving;
- Use the parking support only on flat hard ground;
- Make a transfer to the Bi-Go only when it is in the parking position;
- Use the 6 km/h mode on the sidewalk; the 12 km/h mode only on bike paths;
- Do not clean the Bi-Go with a pressure washer or hard water jet;
- Do not use aggressive cleaning agents;
- Do not use the Bi-Go during thunderstorms;
- Make sure the Bi-Go is serviced on time (after 1500 km or after 1 year).

Warning! During maintenance or repair work, the power supply should always be switched off. **Tip:** If the safety aspects are unclear to you, ask the manufacturer or your supplier for an explanation.

Residual Risks

Residual risks are risks that cannot be eliminated. For example, not all hazardous parts can be shielded because they must remain accessible for operation or maintenance. Risks are minimized when safety regulations are followed.

- If the Bi-Go is not properly secured during transport it could shift resulting in bone fractures or entrapment;

EN

- The Bi-Go may tip over during use due to improper use resulting in broken bones, injury or entrapment;
- The Bi-Go may tip over during use, failure to wear a helmet may result in brain injury or even death;
- It is strictly forbidden to remain in the Bi-Go during transport. Doing so may result in bone fractures and/or entrapment or even death in the event of an accident;
- Lightning strikes are possible during use, resulting in death;
- At a slope angle steeper than 15%, the Bi-Go becomes uncontrollable, there is a safety built in that the Bi-Go cannot go faster and faster, but the danger is only avoided when the slope angle is less than 15% again. A fall from riding on a too steep slope could result in brain damage or, in extreme cases, death;
- When participating in traffic on public roads, it is possible to be involved in an accident, resulting in injury or death.

NL

DE

Technical specifications

- Maximum speeds of 6 km/h or 12 km/h with corresponding steering sensitivity;
- Maximum load capacity 110 kg;
- Dimensions H 64 cm x W 63 cm x L 60 cm;
- Weight 75 kg;
- 2 Lithium-ion batteries;
- Range: up to 30 kilometers, depending on surface, weight and use;
- 2 electric motors;
- 35 cm thermoplastic wheels reinforced with glass fibers;
- 48 cm diameter tires;
- Distance from ground: 9 cm;
- Backrest is foldable;
- Adjustable seat;
- Adjustable footrest.

For specifications of the Segway and batteries, please refer to the Segway User Manual provided.

Warning! Never change the settings of your Bi-Go yourself regarding seat or backrest. Upon delivery of the Bi-Go a careful fitting has been carried out by your supplier with which the seat and backrest are set for the rider. If unauthorized changes are made to these settings, safe operation of the vehicle is no longer guaranteed.

EN

NL

DE

Operation

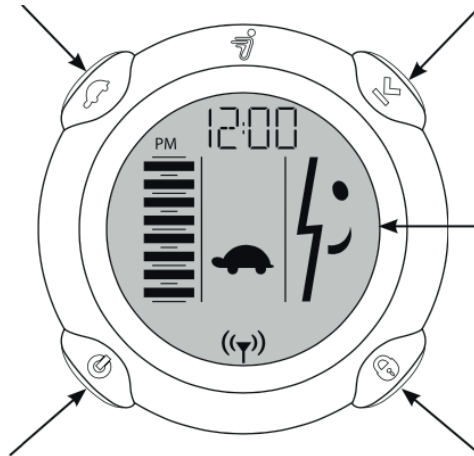
InfoKey

Speed limitation

Switching between the 2 speeds. With the turtle sign 6 km/h; without 12 km/h.

Information display

Change, time, speed, distance traveled or average speed.

Face icon

A series of face icons indicate the status of the Bi-Go; happy, sad, or neutral face depending on the current status.

On-off-Standby

Short press: switch between balancing and non-balancing mode
Long press: switch system switch off completely.

Theft alarm

Activate en deactivate.

InfoKey display symbols

Battery level of the Bi-Go.

Rider sensors are not enabled.

Connection between Bi-Go and InfoKey Controller present.

General warning, reduce speed until the icon disappears.

Service indication, when persistent problems, contact your supplier.

Battery level of the InfoKey Controller low, replace button cell battery and spare battery.

Use

EN In order for you to use the Bi-Go as safely and comfortably as possible, you received explanations and instructions upon delivery of the product. Follow these instructions and the instructions in this manual carefully and ensure proper use of the Bi-Go under all conditions. Please consult your supplier at all times if you have any questions or are unclear regarding the use of your Bi-Go. In addition, we recommend that you wear a helmet during outdoor use.

Seating

NL When you want to take a seat on the Bi-Go, first remove the handlebar. This is done by lifting the bar out of the Bi-Go's holder. Now you can easily transfer to the Bi-Go.

DE Next, place the handlebar back into the holder on the Bi-Go. When you are on the Bi-Go and want to ride, turn the Bi-Go on by pressing the on/off button on the InfoKey. When it is on, you will see the battery level on the InfoKey. You now have the option to choose between speeds of 6 km/h and 12 km/h. When you want to drive six kilometers per hour, make sure the turtle symbol is visible on the InfoKey.

When you are ready to leave, the parking supports must be retracted. You do this by pushing the lever forward. During this procedure, the Bi-Go immediately engages the balancing mode, leaving you almost imperceptibly alone on the two wheels. You are now ready to drive.

Exit/Turn off

When you want to exit the Bi-Go, first make sure the Bi-Go is completely stationary. Then pull the parking lever that operates the legs backwards (never do this while driving!). Make sure the Bi-Go is on a flat and hard surface. When the parking struts are on the ground you will hear a "bleep" sound as a check and you can use the on and off button to take the Bi-Go out of balancing mode. If you then press the on and off button for more than 2 seconds, the Bi-Go switches off completely. To dismount, you can now remove the handlebar.

Parking

If you wish to park the Bi-Go (temporarily), you can pull the handle backwards so that the parking supports fold down. The Bi-Go is now stabilized on the ground. As soon as the parking supports touch the ground, you will hear a control sound ('bleep') from the Segway PT and you must briefly press the on and off button on the InfoKey to deactivate the balancing mode.

As a check, the InfoKey display no longer shows a 'smiley face'. The Bi-Go is now not yet deactivated but it is deactivated (standby) creating a stable parking position. This is important to prevent possible 'crawling' of the vehicle on slippery surfaces.

The latter can occur when the parking supports are on the ground and the Segway PT is still in balancing mode. Please note, in this standby mode, the Bi-Go automatically shuts down after a period of time (usually 15 minutes). So make sure that the Bi-Go is still on before you ride again. Otherwise you will need to turn the Bi-Go on again before you can drive.

Warning! Never use the parking lever when you are still at speed!

Driving

While riding the Bi-Go, you should closely follow the training and instructions you received upon delivery of your Bi-Go. In addition, you should carefully build up experience in practice so that good control, even during unexpected situations, is obtained.

The grip of the tires with the road is crucial to the proper functioning of the Bi-Go. Always keep a close eye on the conditions of the surface you are driving on and stop if there is any doubt about the grip under your tires.

Operating the Bi-Go depends in part on your physical capabilities. Only ride the Bi-Go when you can move your upper body well and unhindered. Always adjust your speed to the conditions and be able to come to a stop in time. Use common sense when facing obstacles in practice and choose the easiest route if possible. Thresholds higher than 5 centimeters should always be avoided.

In case of doubt or questions, contact your supplier for explanations, information or additional training if necessary.

Safety systems

The Bi-Go has a number of safety systems as described below. In all situations, if in doubt, stop and park the vehicle immediately. In that case, contact your supplier immediately.

Stickshake

The 'stickshake' occurs in situations such as extreme driving behavior, reversing too fast or too long and wanting to sit on the Bi-Go when this is not possible. In case of 'stickshake', the Bi-Go vibrates and makes a growling sound. When this occurs while driving, brake immediately and check the InfoKey for any error message. When it is not there, wait 5 seconds and then drive slowly. Memorize any error code and contact your supplier if this problem keeps recurring. If the "stickshake" occurs while you are trying to sit on the Bi-Go, check that the Bi-Go is on a level surface and the handlebar is in a stable position. You cannot ride off now, the stickshake must be remedied before you can use the balancing technique and ride off safely.

Speed limiter

EN The Bi-Go has two speed limits, 6 mph or 12 mph. On sidewalks and walkways, you are allowed to drive 6 km/h. If you want to drive 12 km/h, deactivate the turtle mode via the menu button . Operating the InfoKey is only possible when you take the Bi-Go out of balancing mode (stationary and with the parking supports on the ground).

Emergency procedure

NL When the battery is dead or a malfunction is detected, the Safety Shutdown procedure will start automatically. This will cause the Bi-Go to slow down automatically. Be sure to come to a complete stop as quickly as possible. At that moment, it is important that you park yourself as quickly as possible by pulling the parking lever backwards, because the Bi-Go will shut down within 10 seconds Practice this procedure regularly so that you can react appropriately if necessary. This will help you avoid unsafe situations (such as falling over). Check for an error message on the InfoKey and remember any error code. Always contact your supplier after a DE Safety Shutdown procedure, it is not safe to continue driving. If necessary, consult with your supplier to see if the Bi-Go can be turned back on and, if necessary, used carefully at low speed to get further/home.

Maintenance

Charging

The Bi-Go uses a Segway Base with 2 Li-ion batteries. The batteries require special maintenance and use. To ensure the life of the batteries, we recommend that you handle and follow this manual.

- To keep the batteries of the Bi-Go in optimal condition, it is important that the Bi-Go is always on the charger at times when you are not using it;
- If you are not using the Bi-Go at all for an extended period of time, charge the battery for 7 hours once every 2 weeks.
- The Bi-Go is equipped with a 220V mains power connection;
- Always plug into the Bi-Go first and then plug into the wall outlet;
- On the back of the Bi-Go is the connection point for the charging cable. Both batteries are charged simultaneously;
- The average charging time of the Bi-Go is 7 hours, at this charging time the batteries are 100% charged.

Battery maintenance

When you have completely drained the batteries, and given the Bi-Go a Safety Shut-down; we recommend that you wait one hour before you start charging the Bi-Go. It is recommended that you change the batteries annually. That is, that you place the rear battery at the front and the front battery at the rear. This can of course be performed during the annual service by your supplier.

Maintenance tips

To keep your Bi-Go in optimal condition, keep the following in mind:

- Place the Bi-Go in a safe environment with a temperature of at least 10 °C.
- Fully charge the battery of the Bi-Go at least once a month.
- Maintain the correct tire pressure (1.0 bar).
- Clean your Bi-Go regularly with a damp cloth.

Service

For service, maintenance and repairs to your Bi-Go, please contact your supplier's Service Center.

EN

Adjustment of the Bi-Go

For proper and safe use of your Bi-Go, correct adjustment of your Bi-Go is crucial. The Bi-Go is set specifically for your needs and body. You are not allowed to make any adjustments without consulting your supplier. A modification or adjustment may affect the balance function of the Bi-Go. Making your own adjustments to the Bi-Go hinders safe use and proper operation and can lead to dangerous situations. If you do make adjustments without consultation, the warranty will expire and Mo-biel BV or the supplier will not be held liable.

NL

First check-up

For a first check-up after delivery it is advised to schedule an appointment at your supplier's Service Center after 1500 kilometers or after 1 year with your Bi-Go. This appointment can be scheduled by phone or email. We will determine during this first service, together with you, whether your fit is still completely satisfactory. In addition, the Bi-Go will be checked on a number of crucial points so that you can continue to use your Bi-Go carefree.

DE

Service

Every year or after 1500 kilometers the Bi-Go needs to be serviced. During this service the Bi-Go, the batteries and the Segway base will be disassembled, cleaned, lubricated and adjusted. Wear parts such as the elastomers, (if necessary also other parts), will be replaced. This service is important to ensure your riding enjoyment and to prevent damage to rotating parts. Your service can be performed at your supplier's Service Center.

Tire pressure

The pressure of your standard Bi-Go tires should be 1 Bar. It is important to check your tires regularly and keep them inflated. Too low pressure can result in malfunctions, reduced range and excessive wear. Too high pressure will cause vibrations and greatly affect the riding comfort of your Bi-Go.

The tire valves are located on the inside of the wheel and can be accessed through the underside of the vehicle. If in doubt or unclear, ask your supplier about the correct procedure.

Liability

This operating manual must be used according to the nationally applicable legislation. It is to be considered a part of the Bi-Go and, as required by applicable legislation, must be kept for consultation until its final decommissioning.

This operating manual must be kept by the owner or user in a safe, dry place. It must always be available for consultation.

The published data are based on the latest information. They are provided subject to subsequent changes.

Mobiel BV retains the right, at any time, to change the construction and/or design of its products without the obligation to adjust earlier deliveries accordingly.

The user manual was written according to the NEN standard 5509 and the directives 2006/42/EC and 93/42/EEC were consulted during the design.

In case of damage, the user can request a new copy of this manual from their supplier.

Anyone using the Bi-Go must be aware of the safety regulations applicable.

Mobiel BV/your supplier rejects all responsibility and liability in case of:

- The non-legitimate use of the Bi-Go;
- Failure to observe instructions described in the manual;
- Making changes to the mechanism without written permission;
- The use of the Bi-Go by people who have not received instructions from the supplier;
- Failure to use within the established operating limits;
- Remaining seated in the Bi-Go during transportation by wheelchair cab, for example;
- Failure to perform timely maintenance;
- Performing a repair without using original spare parts;
- Failure to have service performed annually or at 1500 km.

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